

MediPaw® Healing Slim Boot Measurement Chart




1

Print & verify the chart:

- Print the Sizing Chart at 100% size.
- Verify the accuracy of this chart with a ruler before measuring.

2

Measure your patient's paw:

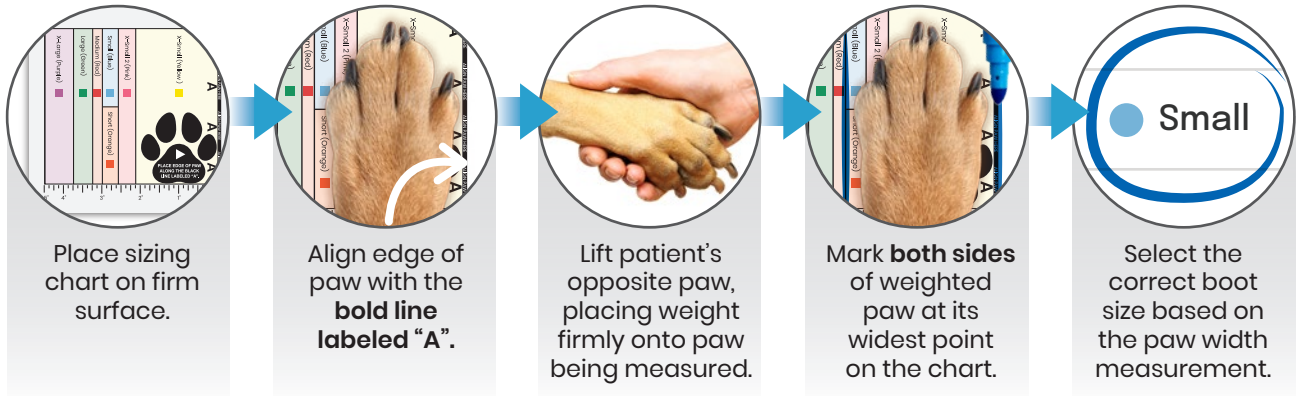

Paw width	Boot height	Boot size	Paw width	Boot height	Boot size
2 in (5.08 cm)	6 in (15.24 cm)	X Small	3 in (7.62 cm)	10.5 in (26.67 cm)	Small
2.5 in (6.35 cm)	8.25 in (20.955 cm)	X Small 2	3.25 in (8.255 cm)	14.25 in (36.195 cm)	Medium
2.5 in (6.35 cm)	7.25 in (18.415 cm)	Short	3.75 in (9.525 cm)	19.75 in (50.165 cm)	Large
			4.5 in (11.43 cm)	25.5 in (64.77 cm)	XL

MEASUREMENT TIPS:

- Measure the width of the paw and the bandage (if applicable) to ensure a correct fit.
- If you can't print the chart, draw a line on either side of the patient's on blank paper, measure the width and refer to the chart above.

Questions?

Please contact us!
 info@medivetproducts.com
 P; 800.245.3413
 F: 646.783.3808

www.medivetproducts.com

Protective Book
 U.S. Patent No. D747,048 S

